


Dr. Becker Full Mouth Speculum, modified by Dr. Stelzer



complete version only \$1325.00

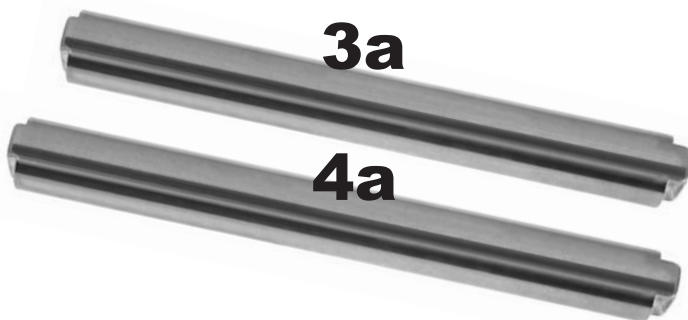

Instructions for assemblage

Before assembling the incisor plates (3 and 4) make sure the two main pieces are opened to the same degree. Use the large set screws (arrows) on the main pieces to change opening.

The round gum pieces (3a and 4a) can be used instead of the incisor plates for incisor work. Remove side screws from incisor plate 3 and 4 and assemble the incisor plates and the main piece as shown in top picture. Tighten the screws using the hex screw driver.

Caution: there are numbers on both the incisor plates and main pieces. Make sure these numbers correspond when assembling the speculum.

Remarks regarding proper use of the speculum

Translated from the instructions 2003; recommendations by the German manufacturer.

Before each use: Make sure both incisor plates are securely attached to the main pieces. Make sure the screws are not loose.

To open/close the speculum: turn the two large set screws always at the same time. See label on screws for turning direction.

Make sure the horse is adequately sedated when using the speculum. Avoid excessive biting on the speculum. Consider general anesthesia in large draft horses.

Clean the threads of the two main pieces after each use. Use lubricants as needed to assure impeccable use.

Do not hesitate to contact Swissvet if you have any questions or concerns.

made in Germany - distributed within the US and Canada by Swissvet Veterinary Products.
www.swissvet.com 1-877 794 7735 mail@swissvet.com