



Horse Dental Equipment




Distributed by
SWISSVET VETERINARY PRODUCTS
3839 Dellwood Drive
Knoxville, TN 37919
877 794 7735 toll free - 865 540 8899 phone
865 522 7424 fax www.powerdentistry.com

■ Put the bearings N°7 (drill side) with a little bit of grease N°10 on.

■ Put the seal N° 8 on the axle N° 6 (open side outside) & put the seal on (always drill side).



■ Lubricate & put the seal on (always drill side) : it allows the seals to stay straight & waterproof.

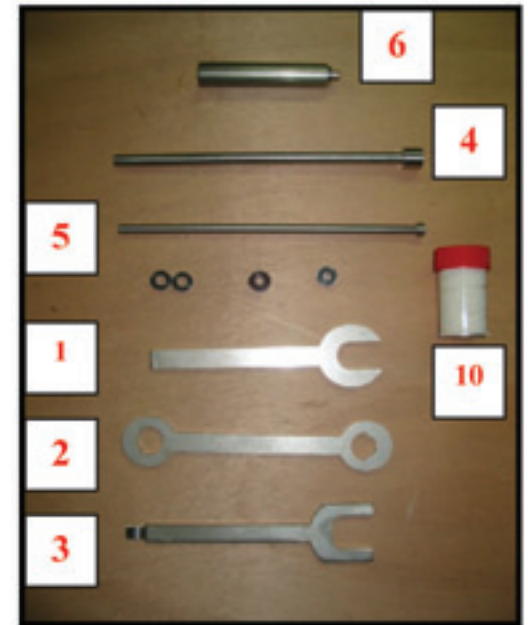


Then, put the drill on body N° 12 & put the bearing N° 9 on the driver screw N° 11.

Screw it on the diamond axle (lightly tighten)

Screw the screw N° 13 & the protection.

Never use an extension to tighten the protection & screw N°13.

WARNING : for the mandibular drill (F2), you must put the protection on the diamond axle before you put it on body N° 12.

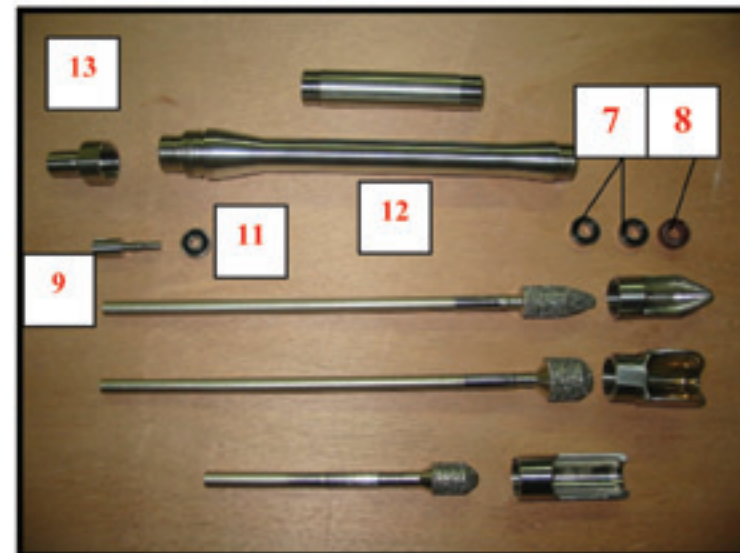



- Unscrew the nut N° 13 (cable side) with spanner N° 1.
- Unscrew the protection with spanner N° 2 or N° 3.

- Bring out the 2 bearings & seal with the axle N°4.



- Block the head with a plier & unscrew the driver screw.



- Bring out the bearing (cable side) with the axle N° 5.



- Bring out the drill.



Before the reassembly, take care to clean all the elements.